



Tap Shaft 00-8790-005-30

Assembly/Disassembly
Instructions



Disassembly Instructions for Tap Shaft

1. Unthread the outer collar **1** from the inner collet **2**. There will be slight resistance during disassembly.
2. End of disassembly.

Note: The outer collar must be disassembled from the inner collet for cleaning and sterilization

Assembly Instructions for Tap Shaft

1. Thread the outer collar **1** onto the inner collet **2**. There will be slight resistance during assembly.
2. End of assembly.



+H124970000300001/\$100610F10G

97-0000-300-00 1003-H23 Printed in USA ©2010 Zimmer, Inc.

