




Zimmer®
**Anatomical Shoulder™
Inverse/Reverse
Suction Device**
Assembly/Disassembly
Instructions

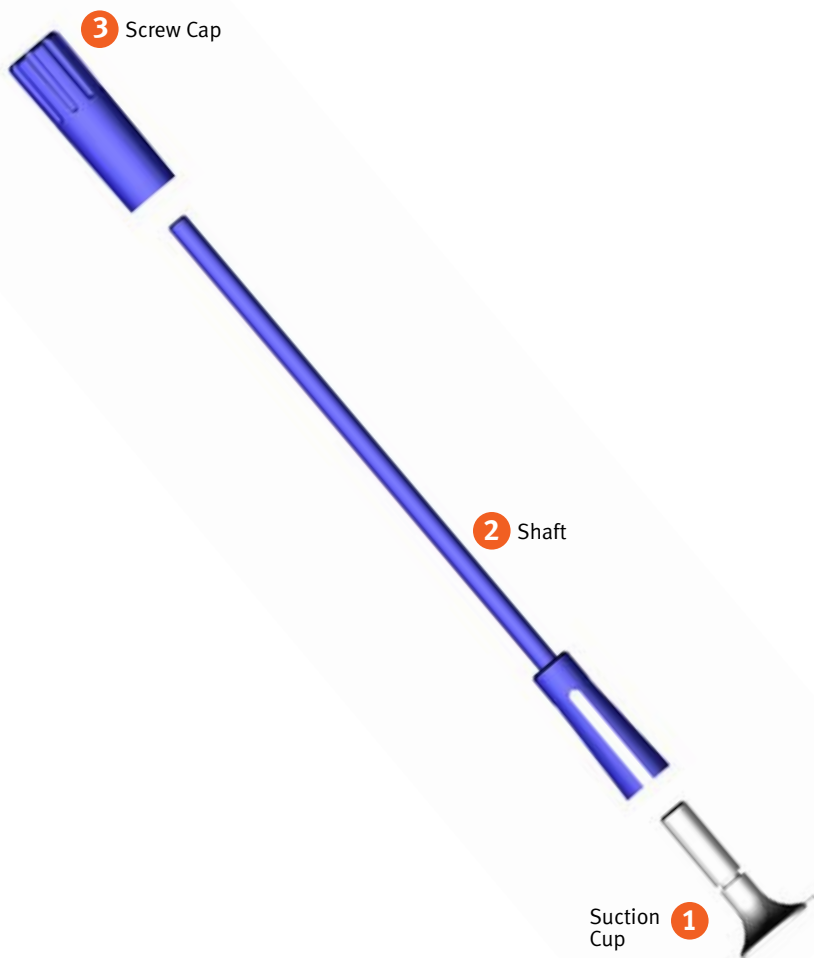

Disassembly Instructions for 01.04239.850

1. Unscrew the Screw Cap **3** and
2. slide off of the shaft **2**.
3. Pull the Suction Cup **1** from the shaft **2**.
4. End of disassembly.

Note: The Suction Cup for the Glensphere must be completely disassembled for cleaning and steam sterilization.

Assembly Instructions for 01.04239.850

1. Push the Suction Cup **1** into the Shaft **2** until the Shaft's **2** fingers are in the Suction Cup's **1** groove.
2. Slide the Screw Cap **3** onto the Shaft **2** with the large hole facing the Suction Cup **1**.
3. Screw the Screw Cap **3** completely to the Shaft **2**.
4. End of assembly.



0086

Zimmer, Inc.
1800 West Center Street
Warsaw, Indiana 46580
USA



Zimmer U.K. Ltd.
9 Lancaster Place
South Marston Park
Swindon, SN3 4FP, UK

©2008 Zimmer, Inc.



zimmer
Confidence in your hands®